unique people unique beds

www.wonderlandbeds.com

## Wonderland **322**

NO

Regulerbar seng

DK

Regulerbar seng

SE

Reglerbar säng

FI

Säätösänky

NL

Verstelbaar bed

DE

Das justierbare Bett

GB

Adjustable bed

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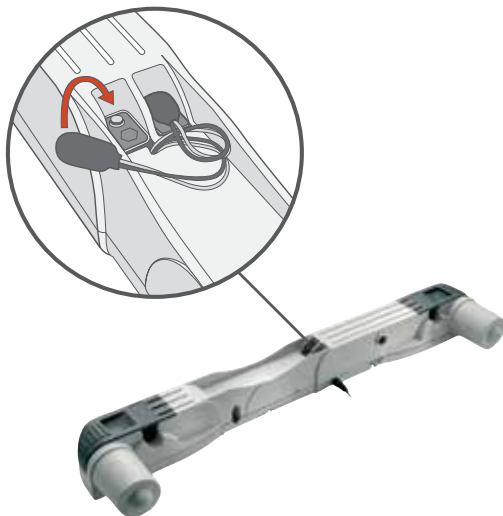
## Emergency power battery

The bed is delivered fully assembled, and the battery should already be connected to the motor. The batteries have a normal life expectancy of one year, after which time they must be replaced. To change the batteries, do as follows: (See illustration.)

- 01** Remove the seal on the battery, if necessary.
- 02** Attach clamps to batteries.
- 03** Insert batteries.

## Remove transport safety ties

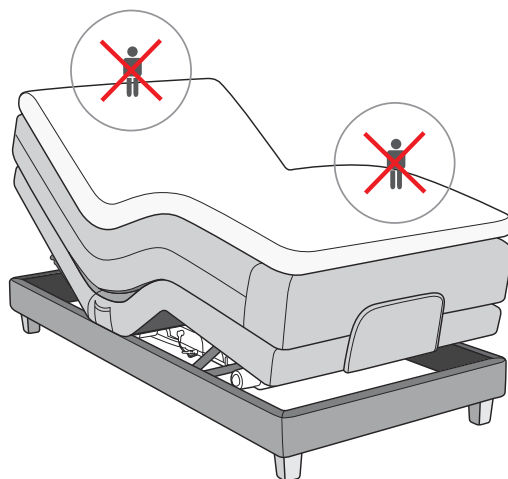
For safe transport, the bed has been secured with ties at the top and bottom. As soon as you have attached the legs and put the bed down with the right side up, you can remove these ties.

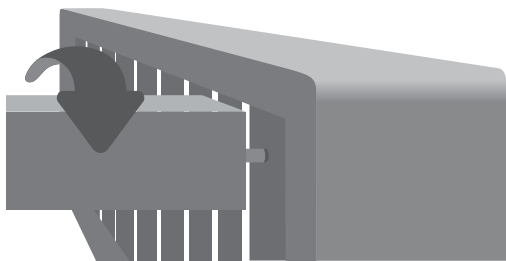
## Power

Plug the cable into a 220V outlet. The bed will not have power until you activate it with the remote control.


## Important!

Whenever the bed is NOT in a horizontal position, you must never sit or stand at the head or foot end of the bed!


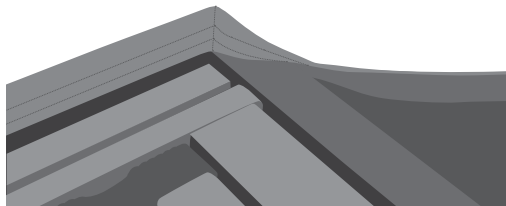
- 01** Fasten the bed legs by screwing them inside the holes underneath the bed


- 02** Put on the textile cover
- a.** The mark "Head End/This side up" should be placed towards the head end of the bed.
  - b.** Make sure the pocket for the remote control is facing upwards and is placed closest to the head end.
  - c.** For the best result, first attach the four corners of the cover on the upper side. Then, start in the middle on the sides attaching the textile cover evenly to the velcro on the bed frame.
  - d.** At last, fasten the textile cover to the velcro on the underside of the bed.

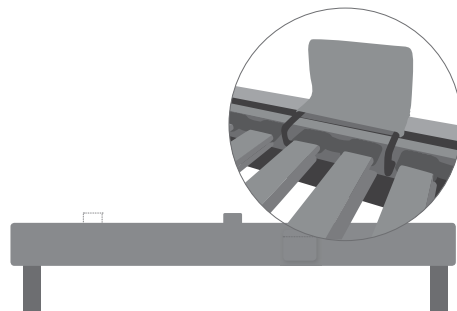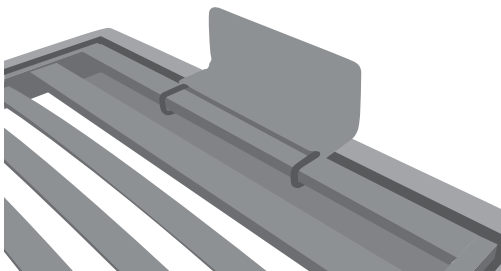- 03** Fasten the side stoppers in the desired positions

- 04** Fasten the end stopper

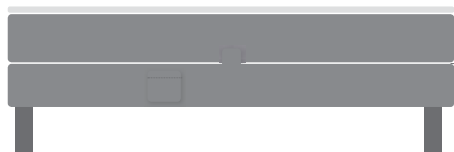- 05** Put on the main mattress and the top mattress

The remote control uses long-range radio frequency. The receiver is integrated in the bed's motor.

Using the remote control you can raise or lower the head or foot end of the bed to a position you choose. You can also raise or lower the head and foot end of the bed simultaneously.

The remote control has irreplaceable, built-in batteries so the remote control must be replaced when the batteries are dead.

Back up

Feet up

Back and feet up

Back down

Feet down

Back and feet down

## Operate two beds with one remote

It's possible to synchronize the beds so that they work with only one remote control. There is a small button on top of the motor. Press this button while pressing down one of the buttons on the remote control. Hold for two seconds. Repeat the procedure on the other bed.

## Power switch

322 is equipped with a power switch in the plug itself. When the bed is not activated with the remote, the power supply to the motor is cut off.

## Emergency lowering of the mattress if the power goes out

The bed is equipped with an emergency battery. If the power goes out, you can still lower the bed into a horizontal position.

## Care and maintenance

To ensure that this adjustable bed remains functional over a long period of time, it requires some simple maintenance in addition to regular cleaning.

You may need to lubricate the moveable metal parts with a liquid lubricant. One of the well-known spray lubricants is recommended.

If you hear a noise/creaking when the bed is activated, you can fix this by tightening the screws. It is normal for wood to contract or expand in different temperatures.

## Adjusting firmness

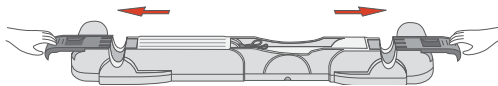
You can adjust the firmness of the hip area by pushing the blue plastic sliders either out towards the sides or in towards the middle. Try different settings until you find the firmness that works for you.

# Replacing the motor. Problems?

## Replacing the motor

Your new adjustable bed has been designed to last for many years. If, however, you should experience problems, most parts are easily replaceable. The main motor is easily replaced:

- 01 Remove the 2 plastic caps holding the motor in place (see illustration). The motor can now be removed.
- 02 Insert the new motor in the right position, and slide the plastic caps back into locking position.

## Problems? The bed does not work

If your bed does not operate correctly, the checklist below can help you find out what is wrong and what to do about it:

---

### Problem **The remote control does not work**

---

- Cause
- Try synchronizing the remote control with the motor (see page 56).
  - Is the plug in place?
  - Does the wall outlet have power?
  - Are the batteries connected to the emergency power battery?
  - Are the batteries in the emergency power function defective? >>

If the advice doesn't work, the remote control is out of battery, and must be replaced with a new remote control.

---

### Problem **The bed cannot be raised or lowered**

---

- Cause
- Is the bed plugged in?
  - Does the wall outlet have power?
  - Unplug the bed and wait a couple of minutes before plugging it in again.
  - Does the remote control have working batteries?

---

### Problem **The bed cannot be lowered on the emergency power supply**

---

- Cause
- Reset the bed by unplugging it. The emergency power function only works when the bed has no other power supply.
  - If the bed still does not work, this is an indication of a problem either in the motor or in the remote control. If you have two beds, you can check this by switching the remote controls.
  - If the bed works with the other remote control, there is a problem with the first remote control. If the bed still does not work after switching the remote controls, it indicates a problem with the motor. Contact the store where you bought your bed.
-

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# 5 YEAR GUARANTEE \*

For material or manufacturing defects.  
Claim does not include normal wear and tear.

**Wonderland AS**

Øran Vest, N-6300 Åndalsnes

Tlf +47 71 22 78 00

Fax +47 71 22 78 01

[info@wonderlandbeds.com](mailto:info@wonderlandbeds.com)

**[www.wonderlandbeds.com](http://www.wonderlandbeds.com)**